

Double Sided Tape Attaching Instructions




1. Wash and exfoliate skin on chest.
2. Wipe back of forms with Skin Tac Wipe (place Skin Tac Wipe in bag for later use)

3. Remove backing paper from one side of tapes and stick one tape to each corner on the back of the form.
*If tapes do not overlap or cover sufficient amount of the form (large forms) you can cut tapes to suit.
Do NOT place tapes around the bottom of the form – as when you stretch up, the form will get pulled off your chest. This will also give them natural “jiggle” when you walk.




4. Making sure the tapes overlap (1-2mm) slightly past the edge of the form (this will stop wrinkling)

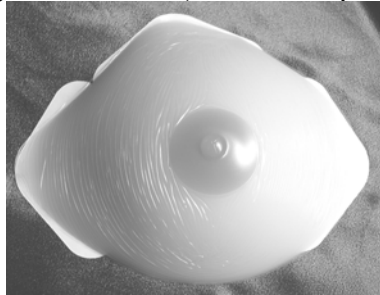


5. Wipe chest area with Skin Tac wipe




6. Remove ONE half of the backing tape from each corner.

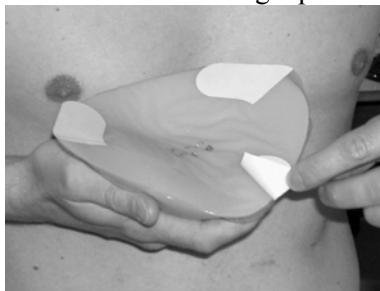

7. Remove the backing tape from the TOP tape.

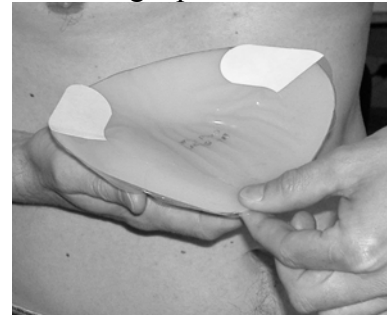


8. Holding the form from the TOP let the form “fall” naturally and line up on chest area and place on chest.

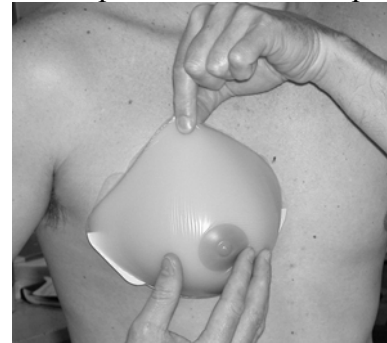

9. Press onto chest firmly.



10. Remove bottom tapes from breast form

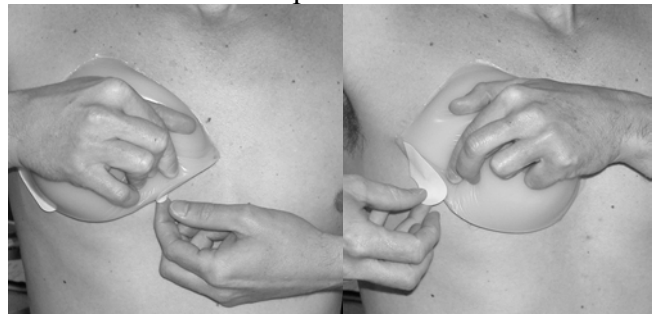

11. Apply pressure to all sides for approx 2 minutes

